## Neoflex Patella 630

### Orpedo

Orthopedics Company KG  
Lietzenburger Str. 75  
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00  
Telefax: +49 (30) 88 71 202-11

Web: [www.orpedo.com](http://www.orpedo.com)  
Mail : [info@orpedo.com](mailto:info@orpedo.com)

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Status: 04/2015

## Specification

The knee support Neoflex Patella 630 gives relief and support to the kneecap. The integrated Pad increases the compression to the patella tendon and therefore supports the kneecap. The perforated Neoprene keeps the body heat. It is moisture repellent, breathable, eudermic. Neoprene provides high wearing comfort.

## Indication

- pain in the front of the knee
- tendinitis beneath the kneecap
- muscular dystrophy
- tendopathy
- chondropathy and arthrosis

## Contraindication

Unknown.

## Instructions

Place pressure pad beneath the kneecap, tear support around you knee and close with velcro. For the application of this product you have to follow its description and intention.

## Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Neoflex Patella 630. In case of numbness or coldness loosen the support or take it off at once. Ask your physician or orthopaedic specialist to check the fit and adjustment. Do not wear the support while sleeping without consulting your physician first.

## Washing instruction (30° )

The support may be washed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

## General Information

Neoprene has isolating and multiple elastic qualities. It keeps body heat, protects musculature and supports the blood flow. It grants enough pressure and support. Since fast cooling-down is prevented the risk of injuries as e.g. sprains, dislocations and muscle ruptures during sport activities is reduced.

## Color

black