
Digivit PIP

Orpedo

Orthopedics Company KG
Lietzenburger Str. 75
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00
Telefax: +49 (30) 88 71 202-11

Web: www.orpedo.com
Mail : info@orpedo.com

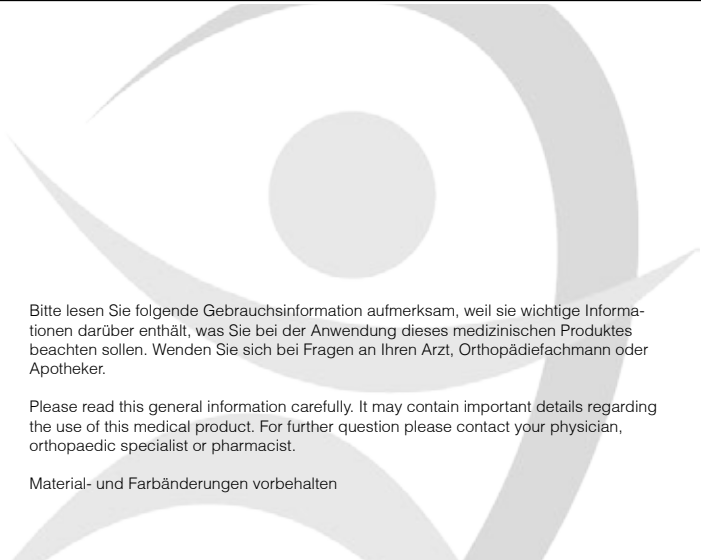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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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INSTRUCTION FOR USE

Status: 04/2015

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The finger splint Digivit PIP offers a Extension of the middle knuckle joint (PIP). The splint is manufactured with plastic and attaches ideal to the finger because of the three-point-principle. The color of the finger splint resembles the skin color which leads to unobtrusive wearing comfort.

Indications:

- inflection of the middle finger hinge

Contraindication

- dermal injury
- open wounds
- swellings
- allergies
- nerv irritation

Instructions

Bed the injured finger on the two curved fixation objects until the flexion of the middle knuckle joint is guaranteed.

Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Digivit PIP.

Material

Thermoplastic

Washing instruction (☞☒☒☒☒)

The support may be hand washed in mild detergent. Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach.

Color

skin