Malleovit Comp 660

Orpedo

Orthopedics Company KG
Lietzenburger Str. 75
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00
Telefax: +49 (30) 88 71 202-11

Web: www.orpedo.com
Mail : info@orpedo.com

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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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Status: 12/2012

Product description

The ankle support Malleovit 660 made of two-ply elastic material is breathable and gives secure support to the ankle and ligaments. It is used for strains, bruises and sprains, preventative or during the treatment. For an increase of compression lateral silicone pads provides a secure positioning of the ankle. Malleovit 660 massages the abdomen, and increases the flow of blood and lymph. It helps to relieve the pain and reduce the swelling. Providing comfortable wearing properties, makes the bandage an optimal aid.

Indication

- irritation after surgery, trauma, overburden of tendons, ligaments (lateral resp.medial) and/or muscular system
- chronic pain in the articulation of the ankle caused by degenerative or rheumatic diseases (arthrosis)
- postoperative situations
- slight injuries of the articulation of the ankle (confusion, distortions, distension)

Contraindication

Unknown

Instructions

Wear Malleovit 660 like a stocking. Make sure pads are located correctly underneath the ankle points on both sides. For the application of this product you have to follow its description and intention.

Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your orthopaedic specialist or pharmacist to ensure the correct fitting of the Malleovit 660. In case of numbness or coldness loosen the support or take it off at once. Ask your physician or orthopaedic specialist to check the fit and adjustment. Do not wear the support while sleeping without consulting your physician first.

Washing instruction (30°)

The support may be handwashed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

Color

black-white, skin