
## Achillovit Comp 670

### **Orpedo**

Orthopedics Company KG  
Lietzenburger Str. 75  
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00  
Telefax: +49 (30) 88 71 202-11

Web: [www.orpedo.com](http://www.orpedo.com)  
Mail : [info@orpedo.com](mailto:info@orpedo.com)

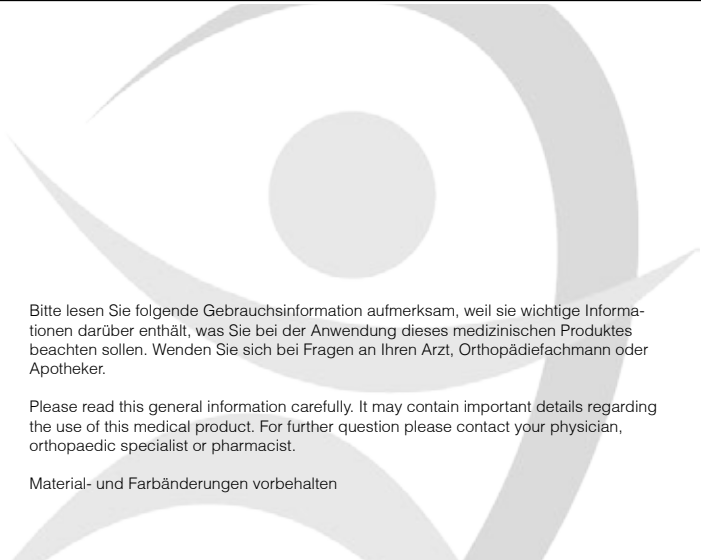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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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Status: 04/2015

## Product description

The Achillovit Comp 670 foot bandage is made of two-ply elastic material, anatomically knitted. A flat knitting technique is used to manufacture it. The bandage has a silicone pad in it. A removable silicone heel wedge is included. The patient can pull out the wedge according to his or her needs, from the pouch that is included for it. The bandage is effective against irritation of the Achilles tendon, and creates a solid hold and stability, due to its anatomic form and material characteristics. The heel wedge provides relief for the tendon during the inflammation phase. The bandage absorbs the force of steps, and provides relief of the Achilles tendon. It relieves pain. The breathable material is particularly friendly to the skin, and, due to the good climatic characteristics, is comfortable. The bandage adapts to the body's movements. Constriction and slippage is prevented.

## Indications

- Achilles tendon tears
- Inflammations
- Post-operative
- Dislocations / lesions / compressions

## Contraindication

Unknown.

## Use instructions

The Achillovit Comp 670 is put on like a sock or stocking. Care should be taken to position the pad correctly. The product is to be used according to the instructions and purposes.

## Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swelling, allergies and other diseases or complication. The first application an adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the achillovit comp 670. In case if numbness or coldness loosen the support or take it off at once. This product has its full effect on the movement. For this reason the bandage has to be taken off in case of long respites or while sleeping. Ask your physician or orthopaedic specialist or pharmacist to check the fit and adjustment.

## Washing instruction (30° )

The support may be handwashed in mild detergent at a temperature of 30° C (86° F). Do not use fabric softener because this affect the compression quality. Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry.