## Cervipol Soft 500

### **Orpedo**

Orthopedics Company KG  
Lietzenburger Str. 75  
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00  
Telefax: +49 (30) 88 71 202-11

Web: [www.orpedo.com](http://www.orpedo.com)  
Mail : [info@orpedo.com](mailto:info@orpedo.com)

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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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Status: 04/2015

## Specifications

The Cervipol Soft 500 is made of foam-material with a natural cover made out of bamboo. It is eudermic and provides high wearing comfort. The polyethylene reinforcement grants stability for the cervical spine.

## Indication

- distortion of cervical spine (including whiplash)
- contusion of cervical spine
- severe pains
- degenerative deformans
- inflammation
- nape-shoulder-arm-syndrome
- postoperative

## Contraindication

Unknown.

## Instructions

Place the cut-out under the chin and close with velcro in the back of the neck. For the application of this product you have to follow its description and intention.

## Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Cerviplas soft 500.

## Washing instruction (30°

The support may be washed in mild detergent at a temperature of 30° C (86° F). Avoid wringing as well as direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

## Colour

beige