
## Lumbovit 300b

### **Orpedo**

Orthopedics Company KG  
Lietzenburger Str. 75  
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00  
Telefax: +49 (30) 88 71 202-11

Web: [www.orpedo.com](http://www.orpedo.com)  
Mail : [info@orpedo.com](mailto:info@orpedo.com)

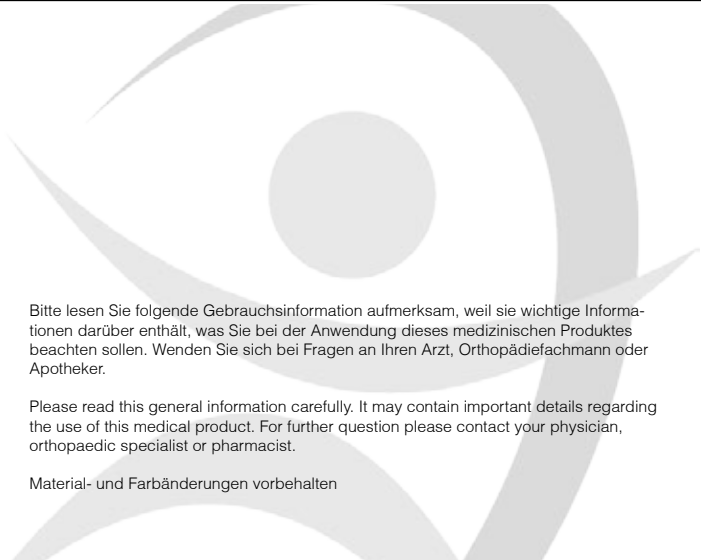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Status: 04/2015

## Product description

The Lumbovit 300b made of high-quality perforated neoprene supports the lower back and relieves pain and prevents further ailments. The support spends soothing therapeutic warmth. The additional pull straps with elastic bands allow for individually adjustable compression and thus variable adjustment of stabilization. The perforated Neoprene keeps the body heat. It is moisture repellent, breathable, eudermic Neoprene provides high wearing comfort.

## Indications

- heavy lumbago
- ischialgia
- relief of the spinal column
- rheumatic pains
- conservative and postoperative with
- spondylosis
- spinal canal stenosis

## Contraindication

Unknown.

## Instructions

Wrap the support around the waist and close with buckles parallelly.

For the application of this product you have to follow its description and intention.

## Notice

Please contact your physician or pharmacist in any of the following cases: Severe injuries, open wounds, swellings, allergies and other diseases or complication. Since the supports have to be customised the first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Lumbovit 300. The supports of the Lumbovit 300 can be customised mechanically to the anatomy of the patient. Do not use ointment while wearing the support.

## Material composition

Inside: 100% neoprene

Outside: 100% nylon

## Washing instruction

After removal of the stays the support may be washed at 30° C in mild detergent at temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

## General Information

Neoprene has isolating and multiple elastic qualities. It keeps body heat, protects musculature and supports the blood flow. It grants enough pressure and support. Since fast cooling-down is prevented the risk of injuries as e.g. sprains, dislocations and muscle ruptures during sport activities is reduced.

## Color

black