Pollicivit Dynamic 440

Orpedo

Orthopedics Company KG
Lietzenburger Str. 75
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00
Telefax: +49 (30) 88 71 202-11

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Status: 04/2015

Product description

The Pollicivital dynamic 440 is designed in an exactly, to the anatomy of the hand and the thumb fitting way. On the inside, there is an aluminium brace that is anatomically preformed and can be adjusted in a form-fitting manner according to the individual circumstances. Velcro fasteners provide an individual adjustment possibility, an easy handling and a perfect fit of the aid. The thumb orthosis can be used at the right and at the left hand. The aluminium brace that can be easily and completely adjusted in adjacent manner, in connection with a plastic strengthening and the three different sizes make the orthosis be an optimal aid in case of the following diagnosis.

Indications

- degenerative disease
- stabilization of the saddle articulation of the Thumb (principle articulation of the thumb)
- chronic, post-traumatic or postoperative irritation in the area of the saddle articulation of the thumb
- postoperative or conservative care
- prevention against overburden

Contraindication

- dermal injury
- open wounds
- swellings
- allergies
- nerv irritation

Instructions

Put the thumb into the orthosis resp. the orthosis around the hand. Check the formfitting position of the aluminium brace and adjust it according to the individual circumstances. For a perfect position of the thumb, pay attention that the aluminium brace is not hardly bent but fits form-fitting, in a round way and reaching into the corners. Close the Velcro fasteners firmly.

Notice

The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the support. In case of numbness or coldness loosen the support or take it off at once. Ask your physician or orthopaedic specialist to check the fit and adjustment. Do not wear the support while sleeping without consulting your physician first.

Material

100% Polyamid
Innen:
100% Polyethylen

Washing instruction

The support product may be handwashed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach.

Color

grey