Malleo-Support Gel 610

Orpedo

Orthopedics Company KG
Lietzenburger Str. 75
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00
Telefax: +49 (30) 88 71 202-11

Web: www.orpedo.com
Mail : info@orpedo.com

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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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Polyacetal, Nylon , TPU-Material (Polyurethan) , Polyester-Schaum

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Status: 07/2015

Product description

The gel padded system with two rigid plastic moulds provides optimal positioning of the ankle joint. It provides ligament support and can be individually adjusted by Velcro® tapes. Sideways motion is only slightly possible and rolling motions are only restricted possible. The Malleo-Support Gel 610 is ideal for the prophylaxis during sport activities and for the rehabilitation of sport injuries. The textile covering surface of the air chambers provides high wearing comfort reducing the sweat forming. The Gel chambers adjust to the swellings heaviness and guarantee an enjoyable wearing comfort. The Gel cushion can be used for cooling. Sweat will be lowered.

Indications

- Fibula ligament injuries
- Distortion
- Postoperative
- Inflammation/swellings
- Lateral rotation trauma
- Contusion
- Degenerative deformations

Contraindications

- open wounds
- allergies
- nerve irritation

Instructions

Open the Velcro® tapes and pull apart the plastic moulds. Put the foot into the ankle support and place the plastic moulds of the Malleo-Support Gel 610 on the right and left side corresponding to the label int. and ext. for each side. The ankles are now on the hole of

Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your orthopaedic specialist or pharmacist to ensure the correct fitting of the Malle-Support Gel 610. In case of numbness or coldness loosen the support or take it off at once. Ask your physician or orthopaedic specialist to check the fit and adjustment. Do not wear the support while sleeping without consulting your physician first.

Washing instruction (30°

The support may be handwashed in mild detergent at a temperature of 30° C (86° F). Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

Size

universal

Color

black