
## Hallux Valgus Night

### **Orpedo**

Orthopedics Company KG  
Lietzenburger Str. 75  
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00  
Telefax: +49 (30) 88 71 202-11

Web: [www.orpedo.com](http://www.orpedo.com)  
Mail : [info@orpedo.com](mailto:info@orpedo.com)

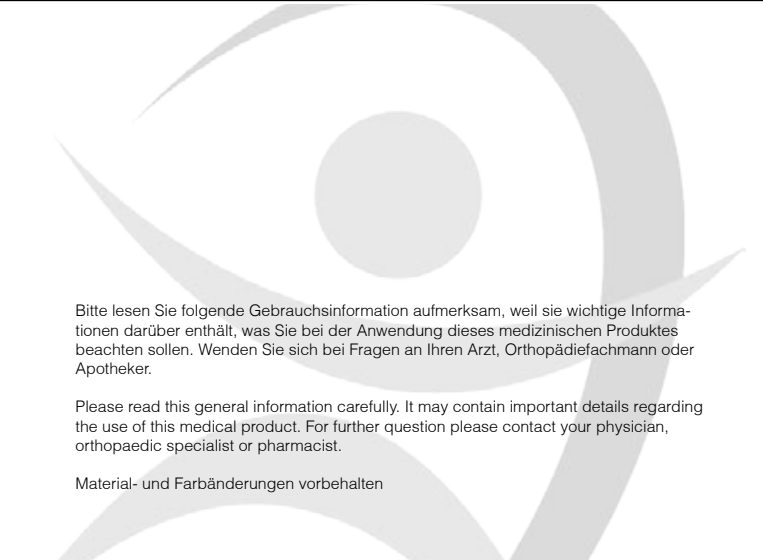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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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Status: 04/2015

## Product description

The splint of Hallux Valgus Night made of thermally adjustable plastic material corrects false position of the big toe and ball of the foot. The soft padded area of the big toe and of the ball of the foot provides high wearing comfort. The Velcro® closure enables an individual correction and adjustment.

## Indications

- Hallux Valgus
- ball formation of the big toe
- false position and deformation of the big toe
- conservative and postoperative therapy

## Contraindication

Unknown.

## Instructions

Take the bandage at the upper ends of the spirals and pull it over the knee. The silicone ring must tightly enclose the patella. Because of the movement of the soft pads, a reconnection might be necessary.

Use the product according to the indications and the purpose.

## Notice

Please contact your physician or pharmacist in any of the following cases: Severe injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Hallux Valgus Night. In case of numbness or coldness loosen the support or take it off at once. Ask your physician or orthopaedic specialist to check the fit and adjustment.

## Washing instruction (30°

The splint may be cleaned with a sponge and mild detergent. For drying use a towel. Do not dry in direct heat (e.g. sunlight, radiator, drier, etc.). Do not bleach or use washing sodas. Do not iron or dry clean.

## Material

Nylon  
Plastazote®  
Synthetic leather

## Color

skin-coloured

## Size

universal