Epivit Flex 500

Orpedo

Orthopedics Company KG
Lietzenburger Str. 75
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00
Telefax: +49 (30) 88 71 202-11

Web: www.orpedo.com
Mail : info@orpedo.com

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Please read this general information carefully. It may contain important details regarding the use of this medical product. For further question please contact your physician, orthopaedic specialist or pharmacist.

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Status: 05/2015

Product description

The elbow bandage Epivit Flex 500 is an epicondylitis-orthesis, which is universally usable due to the individual adaptable truss pad. The smooth adjustable velcro causes a compression on the forearm musculature with reduction of the tension in the enthesis. The orthesis is made by skin friendly material so it offers a high wearing comfort.

Indications

- epicondylitis humeri ulnaris
- epicondylitis humeri radialis

Contraindications

- skin injury
- open wounds
- swellings
- allergies
- nerve irritations

Instructions

apply the epicondylitis orthesis by attaching the velcro to the truss pad for the left or right forearm.

A correction afterwards can be necessary. The orthesis is recommended for its purpose.

Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swellings, allergies and other diseases or complication. The first application and adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the Epivit Flex 500. In case of numbness or coldness loosen the support or take it off at once. Ask your physician or orthopaedic specialist to check if you can wear the orthesis while sleeping.

Material composition

100% polyamide

Washing instruction

The bandage may be handwashed in mild detergent at a temperature of 30° C (86° F). To guarantee a long durability of the velcro, please close it before washing. The removable truss pad can also be washed in a mild detergent. Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry clean.

Size

universal

Color

dark grey