Thorax-Abdominivit 680 mit Aloe Vera

Orpedo

Orthopedics Company KG
Lietzenburger Str. 75
D-10719 Berlin

Telefon: +49 (30) 88 71 202-00
Telefax: +49 (30) 88 71 202-11

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Status: 04/2015

Description

The Thorax-Abdominivit 680 is a body bandage with integrated aloe vera microcapsules and silver ions. The bandage enables an optimal support of the back and the abdominal wall, and helps the healing process of post-operative wounds. The Velcro fasteners provide ease of use and a perfect positioning of the product. Its elastic characteristics in the length direction enable the compression to be optimally adapted. Even pressure distribution is ensured. The therapeutic effect is also supported by the integrated aloe vera and the silver ions. Aloe vera and silver are, due to their antiseptic properties, ideal for wound disinfection, and also accelerate cell renewal and the healing process. The slight scent of aloe vera produces a pleasant feeling when the product is being worn.

Indications

- Scar protection
- Umbilical hernias
- Abdominal wall conditions
- After abdominal surgery
- For post-operative support
- Rib fractures
- Hernias

Contraindication

Unknown.

Use instructions

The Thoracic Abdominivit 680 with aloe vera is positioned around the waist. The product is to be used according to the instructions and purposes.

Material composition

Polyamide: 63%
Spandex: 15%
Cotton: 12 %
Polyester: 10%
Silverion
Aloe Vera Micropackages

Notice

Please contact your physician or pharmacist in any of the following cases: severe injuries, open wounds, swelling, allergies and other diseases or complication. The first application an adjustment should be done by your physician, orthopaedic specialist or pharmacist to ensure the correct fitting of the thoracic abdominivit 680 with aloe vera. In case if numbness or coldness loosen the support or take it off at once. This product has its full effect on the movement. For this reason the bandage has to be taken off in case of long respites or while sleeping. Ask your physician or orthopaedic specialist or pharmacist to check the fit and adjustment.

Washing instruction (☹️🚫🧼🧴)

The support may be handwashed in mild detergent at a temperature of 30° C (86° F). Do not use fabric softener because this affect the compression quality. Avoid direct sunlight, heat of radiators, electric iron or the like. Do not bleach or dry.