Orpedo GmbH
Stieffring 2
Berlin D-13627

www.orpedo.com
+49 30 88 71 20 200
info@orpedo.com

Größe / Size	REF	Hals / Neck
S	108-105-1	25 - 32 cm
M	108-105-2	33 - 40 cm
L	108-105-3	41 - 48 cm

CERVIPLAS 111 (TRACHEA)

Hersteller
Manufacturer

Gebrauchsanweisung beachten
Consult instructions for use

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Please read this general information carefully. It contains important details regarding the use of this medical device. For questions, please contact your doctor or medical supply store.

In case any serious incident related with the device occurs, report it to the manufacturer. If the device shows signs of damage, stop using it immediately and dispose of it.

INSTRUCTIONS FOR USE (EN)

CERVIPLAS 111 (TRACHEA) - Immobilization

PRODUCT DESCRIPTION

The device has been designed to immobilize the neck. It is intended to keep the neck in position, ease pain, reduce stress, and support the neck during recovery. The device is easy to care for and offers a high level of wearing comfort thanks to its neck and chin support. There is an opening (50 mm diameter) in the larynx area, e.g., for a tracheostomy tube.

Primary material: Plastazote (polyethylene foam)

This device is intended for single patient multiple use.

INDICATIONS

- post-traumatic conditions (e.g. cervical fractures, sprains, ligamentous injuries)
- post-operative stabilization / protection during rehabilitation
- acute injuries requiring immobilization and rest

CONTRA-INDICATIONS

- Material hypersensitivity
- Open wounds at the neck

APPLICATION

1. Measure Your Neck
 - Measure the circumference of your neck.
 - Select the correct size using the size chart on the packaging.
2. Prepare the Collar
 - Release the Velcro straps of the two half shells.
3. Position the Panels
 - Sit upright with your head still and facing forward.
 - Place the front panel (with the hole) and back panel around your neck.
 - Ensure the rounded edges of both panels face downward.
 - Position the front panel under your chin so the chin sits directly within the groove. Your nose and chin should form a straight line through the center of the collar.
4. Secure the Collar
 - The two halves of the collar should fit snugly to restrict head movement.
 - Use the image on the packaging as a reference for correct positioning.

WARNINGS

- Contact your doctor or your medical supply store if complications arise.
- The first application and adjustment should be done by your doctor or your medical supply store.
- To maintain product safety and effectiveness, follow the device care instructions. Failure to comply may impair the functionality of the orthosis and result in potential health risks. Do not use cleaning methods other than those specified.

CARE

- Clean with a sponge and mild detergent at a maximum temperature of 30°C.
- Dry with a towel and do not dry in direct heat (e.g., sunlight, radiator, etc.).
- Do not iron or dry clean.
- Do not bleach or use washing sodas.

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