Orpedo GmbH  
Stieffring 2  
Berlin D-13627

www.orpedo.com  
+49 30 88 71 20 200  
info@orpedo.com

# LUMBOVIT MULTI FLEX 500

Größe / Size	REF	Brust / Chest
1	428-500-1	65 - 125 cm
2	428-500-2	126 - 150 cm

Hersteller  
Manufacturer

Gebrauchsanweisung beachten  
Consult instructions for use

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Please read this general information carefully. It contains important details regarding the use of this medical device. For questions, please contact your doctor or medical supply store.

In case any serious incident related with the device occurs, report it to the manufacturer. If the device shows signs of damage, stop using it immediately and dispose of it.

# INSTRUCTIONS FOR USE (EN)

## LUMBOVIT MULTI FLEX 500 - Posture correction

### PRODUCT DESCRIPTION

The device has been designed to correct the posture of the lower back. It is intended to ease pain, improve posture, and support healing of the back, trunk or spine. The device can be fitted depending on your recovery. It is a modular device with a textile belt, a front pad, and a back pad. The back pad is adjustable in height to match your anatomy. The back pad is worn with memory foam and breathable Coolmax® fabric.

Primary materials: polyester, polyamide, nylon. Inner material: Coolmax® memory foam  
This device is intended for single patient multiple use.

### INDICATIONS

- post-operative (spinal) support (as prescribed by clinician)
- post-traumatic conditions (e.g. lumbar fractures, muscle/ligament strains, dislocations)
- chronic or acute (back) pain (e.g. lumbosacral radiculopathy, prolapse, spinal stenosis)
- degenerative spinal disorders
- muscular weakness or imbalance requiring posture corrections (support in postural deformities)
- support in case of pain or injury during physical activity or occupational load

### CONTRA-INDICATIONS

- Material hypersensitivity
- Open wounds at the lower back

### APPLICATION

1. Measure Your Waist
  - Measure the circumference of your waist.
  - Select the correct size using the size chart on the packaging.
2. Adjust the Back Pad
  - Set the length of the back pad by pushing the red buttons outward.
  - Slide the panels to the correct length. Then push the red buttons inward.
3. Attach the Foam Pad
  - Place the blue Coolmax® memory foam over the back pad.
  - Secure it to the belt.
4. Position the Belt
  - Wrap the belt around your hips so the back pad is centered on your spine.
  - Optionally, attach the front pad to the end of the belt.
5. Secure the Belt
  - Pull both sides of the belt and fasten them together over your abdomen.
6. Adjust the Tension Straps
  - Pull both small tension straps from the back towards the front.
  - Attach them to the front belt.
  - Tighten evenly for a comfortable, supportive fit.

### WARNINGS

- Contact your doctor or your medical supply store if complications arise.
- The first application and adjustment should be done by your doctor or your medical supply store.
- To maintain product safety and effectiveness, follow the device care instructions. Failure to comply may impair the functionality of the orthosis and result in potential health risks. Do not use cleaning methods other than those specified.

### CARE

- Clean with a mild detergent at a maximum temperature of 30°C.
- Remove the pad and secure all fasteners before washing.
- Ideally clean by hand, otherwise use a laundry net when using the washing machine.
- Dry with a towel and do not dry in direct heat (e.g., sunlight, radiator, etc.).
- Do not iron or dry clean.
- Do not bleach or use washing sodas.

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