Orpedo GmbH  
Stieffring 2  
Berlin D-13627

www.orpedo.com  
+49 30 88 71 20 200  
info@orpedo.com

Größe / Size	REF Short/Kurz	REF Long /Lang	Schuh / Shoe
S	628-605-1	628-606-1	36 - 40
M	628-605-2	628-606-2	41 - 44
L	628-605-3	628-606-3	45 - 47
XL	628-605-4	628-606-4	> 47

# AIR WALKER PEDIVIT 605-606

Hersteller  
Manufacturer

Gebrauchsanweisung beachten  
Consult instructions for use

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Please read this general information carefully. It contains important details regarding the use of this medical device. For questions, please contact your doctor or medical supply store.

In case any serious incident related with the device occurs, report it to the manufacturer. If the device shows signs of damage, stop using it immediately and dispose of it.

# INSTRUCTIONS FOR USE (EN)

## AIR WALKER PEDIVIT 605-606 – Immobilization

### PRODUCT DESCRIPTION

The device has been designed to immobilize the foot and ankle. It is intended to keep the foot and ankle in position, ease pain, reduce stress, and support them during recovery. The device is secured around the foot, ankle and shin with Velcro straps. The toe guard is adjustable and the slightly curved sole supports a natural gait. The soft inner lining is equipped with an integrated air cushion system that allows for adjustable compression and optimal fit. The shorter version, AIR WALKER PEDIVIT 605, extends to mid-calf. The longer version, AIR WALKER PEDIVIT 606, extends to the knee.

Primary materials: polyamide, plastic

This device is designed for use on both the right and left leg.

This device is intended for single patient multiple use.

### INDICATIONS

- post-traumatic conditions (e.g. fractures, ankle sprains, ankle distortion, ligamentous injuries)
- post-operative stabilization / protection during rehabilitation
- acute injuries requiring immobilization and rest (e.g. achilles tendon ruptures)

### CONTRA-INDICATIONS

- Material hypersensitivity
- Open wounds at the affected foot and ankle

### APPLICATION

1. Measure Your Foot
  - Measure the circumference of your foot.
  - Select the correct size using the size chart on the packaging.
2. Prepare the Brace
  - Open all Velcro straps and the inner liner completely.
  - Remove the front hard panel.
3. Position Your Foot
  - Place your foot carefully into the soft inner liner.
  - Ensure your heel is seated firmly at the back of the boot.
4. Adjust the Toe Guard
  - To unlock, turn the toe guard upwards and press on both sides to release.
  - Slide the toe guard to the desired foot length.
  - Ensure the toe guard lies flat after adjustment.
  - If there is extra space inside the brace, insert the provided foam pads where needed.
5. Optional: Add Foam Pads
6. Close the inner liner
7. Attach the Front Panel
  - Place the front hard panel over your foot.
  - Reattach the straps over the ankle first, then over the lower leg.
  - The fit should be snug but not too tight.
8. Inflate the Boot
  - Turn the knob on the front panel to "INFLATE".
  - Pump the black pump until the desired compression is achieved.
9. Deflate the Boot
  - Turn the knob to the "DEFLATE" position to release air.

### WARNINGS

- Contact your doctor or your medical supply store if complications arise.
- The first application and adjustment should be done by your doctor or your medical supply store.
- To maintain product safety and effectiveness, follow the device care instructions. Failure to comply may impair the functionality of the orthosis and result in potential health risks. Do not use cleaning methods other than those specified.

### CARE

- Clean with a mild detergent at a maximum temperature of 30°C.
- Remove the pad and secure all Velcro before washing.
- The hard outer shell must be cleaned with a wet wipe.
- Ideally clean the textile by hand, otherwise use a laundry net when using the washing machine.
- Dry with a towel and do not dry in direct heat (e.g., sunlight, radiator, etc.).
- Do not iron or dry clean.
- Do not bleach or use washing sodas.

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