Orpedo GmbH
Stieffring 2
Berlin D-13627

www.orpedo.com
+49 30 88 71 20 200
info@orpedo.com

Größe / Size	REF 7 cm	REF 9 cm	REF 11 cm	Hals / Neck
S	108-102-7-1	108-102-9-1	108-102-11-1	25 - 32 cm
M	108-102-7-2	108-102-9-2	108-102-11-2	33 - 41 cm
L	108-102-7-3	108-102-9-3	108-102-11-3	42 - 55 cm

CERVIPOL SOFT 500

Hersteller
Manufacturer

Gebrauchsanweisung beachten
Consult instructions for use

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Please read this general information carefully. It contains important details regarding the use of this medical device. For questions, please contact your doctor or medical supply store.

In case any serious incident related with the device occurs, report it to the manufacturer. If the device shows signs of damage, stop using it immediately and dispose of it.

INSTRUCTIONS FOR USE (EN)

CERVIPOL SOFT 500 - Unloading

PRODUCT DESCRIPTION

The device has been designed to relieve the neck. It is intended to reduce stress, ease pain, improve posture, and alignment during recovery or preventive use. The collar is covered with a skin-friendly bamboo yarn. It promotes gentle neck stabilization and limits the range of movement. To make it easy to put on, it is secured with a Velcro strap.

Primary materials: polyethylene foam, bamboo

This device is intended for single patient multiple use.

INDICATIONS

- post-traumatic or post-surgical pain and/or discomfort (e.g. whiplash, acute sprain)
- discomfort or pain caused by degenerative, overuse or insufficiency conditions of the musculoskeletal system (e.g. neck stiffness, cervical disc herniation, cervical radicular syndrome)

CONTRA-INDICATIONS

- Material hypersensitivity
- Open wounds at the neck

APPLICATION

1. Measure Your Neck
 - Measure the circumference and height of your neck.
 - Select the correct size using the size chart on the packaging.
2. Position the Collar
 - Ensure the logo on the Velcro strap is facing outwards and upwards.
 - Place the U-shaped edge centrally under your chin.
3. Secure the Strap
 - Fasten the Velcro strap behind your neck for a snug fit, preventing head movement.

WARNINGS

- Contact your doctor or your medical supply store if complications arise.
- The first application and adjustment should be done by your doctor or your medical supply store.
- To maintain product safety and effectiveness, follow the device care instructions. Failure to comply may impair the functionality of the orthosis and result in potential health risks. Do not use cleaning methods other than those specified.

CARE

- Clean with a mild detergent at a maximum temperature of 30°C.
- Dry with a towel and do not dry in direct heat (e.g., sunlight, radiator, etc.).
- Do not iron or dry clean.
- Do not bleach or use washing sodas.

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