Orpedo GmbH  
Stieffring 2  
Berlin D-13627

www.orpedo.com  
+49 30 88 71 20 200  
info@orpedo.com

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Please read this general information carefully. It contains important details regarding the use of this medical device. For questions, please contact your doctor or medical supply store.

In case any serious incident related with the device occurs, report it to the manufacturer. If the device shows signs of damage, stop using it immediately and dispose of it.

# INSTRUCTIONS FOR USE (EN)

## HUMEROVIT 370 - Unloading

### PRODUCT DESCRIPTION

The device has been designed to relieve the shoulder and elbow. It is intended to reduce stress, ease pain, improve posture, and alignment during recovery or preventive use. The device consists of a padded envelope that holds the upper arm and elbow in a neutral position. The device supports the arm with an over-shoulder sling to secure the arm close to the body. The shoulder strap can be adjusted to optimize positioning for each body and arm length.

Primary materials: foam, polyester

This device is designed for use on both the right and left arm.  
This device is intended for single patient multiple use.

### INDICATIONS

- post-traumatic or post-surgical pain and/or discomfort (e.g. rotator cuff injury, conservative subcapital humerus, olecranon fractures after surgical treatment)
- discomfort or pain caused by degenerative, overuse or insufficiency conditions of the musculoskeletal system (capsular or ligamentous insufficiency)

### CONTRA-INDICATIONS

- Material hypersensitivity
- Open wounds at the affected shoulder and elbow

### APPLICATION

1. Prepare the Sleeve
  - Locate the Velcro strap at the wrist section of the sleeve.
  - Remove and reattach the strap so that the vertical plastic loop faces toward you.
  - Ensure the Velcro ends are secured to the outside surface of the sleeve.
2. Position the Arm
  - Place your affected arm in the sleeve.
  - Pass the strap attached to the elbow section of the sleeve over your non-affected shoulder and thread it through the top plastic loop (from Step 1).
3. Adjust the Arm Angle
  - Pull the strap to tighten until your arm rests at a 90° angle.
  - Secure the strap by attaching it to itself using the Velcro fasteners.
4. Optional: Add Waist Support
  - Take the long additional strap.
  - Thread it through the vertical plastic loop facing you (from Step 1).
  - Wrap the strap around your waist and fasten it with Velcro for extra stability.
5. Optional: Secure your elbow
  - Attach the short strap with 2 Velcro patches around the sleeve, close to your elbow for added support.

### WARNINGS

- Contact your doctor or your medical supply store if complications arise.
- The first application and adjustment should be done by your doctor or your medical supply store.
- To maintain product safety and effectiveness, follow the device care instructions. Failure to comply may impair the functionality of the orthosis and result in potential health risks. Do not use cleaning methods other than those specified.

### CARE

- Clean with a mild detergent at a maximum temperature of 30°C.
- Ideally clean the textile by hand, otherwise use a laundry net when using the washing machine.
- Dry with a towel and do not dry in direct heat (e.g., sunlight, radiator, etc.).
- Do not iron or dry clean.
- Do not bleach or use washing sodas.

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