Orpedo GmbH
Stieffring 2
Berlin D-13627

www.orpedo.com
+49 30 88 71 20 200
info@orpedo.com

PARADISOL 800

Hersteller
Manufacturer

Gebrauchsanweisung beachten
Consult instructions for use

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Please read this general information carefully. It contains important details regarding the use of this medical device. For questions, please contact your doctor or medical supply store.

In case any serious incident related with the device occurs, report it to the manufacturer. If the device shows signs of damage, stop using it immediately and dispose of it.

INSTRUCTIONS FOR USE (EN)

PARADISOL 800 - Unloading

PRODUCT DESCRIPTION

The device has been designed to relieve the abdomen and lower back. It is intended to reduce stress, ease pain, improve posture, and alignment during recovery. The device is a pregnancy belt that lifts and supports the pregnant belly. It reduces strain on the spine and hips. The belt is placed around the lower back and fastened under the belly with adjustable side straps.

Primary materials: elastane, polyester, viscose

This device is intended for single patient multiple use.

INDICATIONS

- post-traumatic or post-surgical pain and/or discomfort
- discomfort or pain caused by degenerative, overuse or insufficiency conditions of the musculoskeletal system (e.g. spinal prolapse, discomfort caused by high bladder pressure)

CONTRA-INDICATIONS

- Material hypersensitivity
- Open wounds at the abdomen and lower back

APPLICATION

1. Select the Belt Size
 - Two soft belts are provided: one small and one large.
 - Choose the size that best fits your abdomen width.
 - Attach the selected belt to one side of the Velcro section on the back brace.
2. Position the Back Brace
 - Place the back brace over your lower back.
 - Ensure it is centred on your spine.
 - Wrap the belt around your lower abdomen and attach it to the Velcro section of the back brace.
3. Adjust the Tension Straps
 - Pull both small tension straps from the back towards the front.
 - Attach them to the front soft belt.
 - Tighten evenly for a comfortable, supportive fit.

WARNINGS

- Contact your doctor or your medical supply store if complications arise.
- The first application and adjustment should be done by your doctor or your medical supply store.
- To maintain product safety and effectiveness, follow the device care instructions. Failure to comply may impair the functionality of the orthosis and result in potential health risks. Do not use cleaning methods other than those specified.

CARE

- Clean with a sponge and mild detergent at a maximum temperature of 30°C.
- Secure all fasteners before washing.
- Dry with a towel and do not dry in direct heat (e.g., sunlight, radiator, etc.).
- Do not iron or dry clean.
- Do not bleach or use washing sodas.

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