Orpedo GmbH  
Stieffring 2  
Berlin D-13627

www.orpedo.com  
+49 30 88 71 20 200  
info@orpedo.com

Größe / Size	REF	Handgelenk / Wrist
S	318-435-1	13 - 16 cm
M	318-435-2	17 - 19 cm
L	318-435-3	20 - 22 cm

## MANU POLLICIVIT 435

Hersteller  
Manufacturer

Gebrauchsanweisung beachten  
Consult instructions for use

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Please read this general information carefully. It contains important details regarding the use of this medical device. For questions, please contact your doctor or medical supply store.

In case any serious incident related with the device occurs, report it to the manufacturer. If the device shows signs of damage, stop using it immediately and dispose of it.

# INSTRUCTIONS FOR USE (EN)

## MANU POLLICIVIT 435 - Motion restriction

### PRODUCT DESCRIPTION

The device has been designed to restrict the motion of the wrist and thumb. It is intended to protect the wrist and thumb, ease pain, and support the wrist and thumb during recovery. An aluminum splint restricts the range of motion of the wrist. The cuff immobilizes the hand, and the Velcro fasteners provide compression and a customized fit.

Primary materials: cotton, polyamide, aluminum

The device is designed for use on the left and right wrist. This device is intended for single patient multiple use.

### INDICATIONS

- post-operative conditions requiring motion restriction
- post-traumatic conditions requiring motion restriction (e.g. ligamentous injuries, nerve lesions, sprains, after cast/plast removal)
- overuse or repetitive strain conditions (e.g. carpal tunnel syndrome,
- degenerative or inflammatory conditions (e.g. arthritis, osteoarthritis)
- support during functional rehabilitation or physical activity

### CONTRA-INDICATIONS

- Material hypersensitivity
- Open wounds at the affected wrist and thumb

### APPLICATION

1. Measure Your Wrist
  - Measure the circumference of your wrist.
  - Select the correct size using the size chart on the packaging.
2. Adjust the Aluminum Splint
  - Remove the aluminum splint from the brace.
  - Shape them to match the natural curve of your wrist or your required resting position.
  - Reinsert the splint into the brace.
3. Put on the Brace
  - Slide your affected hand into the brace.
  - Ensure the splint is positioned on the palm side.
4. Secure the Strap
  - Wrap the strap around your wrist and thumb.
  - Fasten it so it fits snugly, but not too tight.

### WARNINGS

- Contact your doctor or your medical supply store if complications arise.
- The first application and adjustment should be done by your doctor or your medical supply store.
- To maintain product safety and effectiveness, follow the device care instructions. Failure to comply may impair the functionality of the orthosis and result in potential health risks. Do not use cleaning methods other than those specified.

### CARE

- Clean with a mild detergent at a maximum temperature of 30°C.
- Remove the splints before washing.
- Ideally clean the textile by hand, otherwise use a laundry net when using the washing machine.
- Dry with a towel and do not dry in direct heat (e.g., sunlight, radiator, etc.).
- Do not iron or dry clean.
- Do not bleach or use washing sodas.

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