Orpedo GmbH  
Stieffring 2  
Berlin D-13627

www.orpedo.com  
+49 30 88 71 20 200  
info@orpedo.com

## MALLEO-SUPPORT PAD 620

Hersteller  
Manufacturer

Gebrauchsanweisung beachten  
Consult instructions for use

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Please read this general information carefully. It contains important details regarding the use of this medical device. For questions, please contact your doctor or medical supply store.

In case any serious incident related with the device occurs, report it to the manufacturer. If the device shows signs of damage, stop using it immediately and dispose of it.

# INSTRUCTIONS FOR USE (EN)

## MALLEO-SUPPORT PAD 620 - Motion restriction

### PRODUCT DESCRIPTION

The device has been designed to restrict the motion of the ankle. It is intended to protect the ankle, ease pain, and support the ankle during recovery. The device is individually adjustable with Velcro straps for secure fixation and easy application. The padding is covered in fabric and ensures wearing comfort.

Primary materials: polyamide, plastic

The device is designed for use on the left and right ankle. This device is intended for single patient multiple use.

### INDICATIONS

- post-operative conditions requiring motion restriction
- post-traumatic conditions requiring motion restriction (e.g. (fibulotalar) ligamentous injuries, nerve lesions, sprains, contusion)
- overuse or repetitive strain conditions (e.g. supination trauma)
- degenerative or inflammatory conditions (e.g. tendinitis)
- support during functional rehabilitation or physical activity

### CONTRA-INDICATIONS

- Material hypersensitivity
- Open wounds at the affected ankle

### APPLICATION

1. Prepare the Brace
  - Undo all Velcro straps.
  - Position the two ergonomically shaped shells around your ankle.
2. Initial Positioning
  - Loosely wrap the main strap around your ankle.
  - Loosely fasten the uppermost strap.
  - Adjust and secure the strap under your foot to fit your foot width.
3. Final Adjustment
  - Reposition and tighten all straps for a snug, comfortable fit.
  - Use the image on the packaging as a reference for correct positioning.

### WARNINGS

- Contact your doctor or your medical supply store if complications arise.
- The first application and adjustment should be done by your doctor or your medical supply store.
- To maintain product safety and effectiveness, follow the device care instructions. Failure to comply may impair the functionality of the orthosis and result in potential health risks. Do not use cleaning methods other than those specified.

### CARE

- Clean with a mild detergent at a maximum temperature of 30°C.
- Dry with a towel and do not dry in direct heat (e.g., sunlight, radiator, etc.).
- Do not iron or dry clean.
- Do not bleach or use washing sodas.

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