Orpedo GmbH
Stieffring 2
Berlin D-13627

www.orpedo.com
+49 30 88 71 20 200
info@orpedo.com

| Größe / Size | REF | Arm / Arm | Farbe / Color |
|--------------|-----------|------------|---------------|
| S | 218-375-1 | 23 - 27 cm | Grau / Gray |
| M | 218-375-2 | 28 - 31 cm | Grau / Gray |
| L | 218-375-3 | 32 - 34 cm | Grau / Gray |
| XL | 218-375-4 | 35 - 37 cm | Grau / Gray |
| S | 218-376-1 | 23 - 27 cm | Beige / Beige |
| M | 218-376-2 | 28 - 31 cm | Beige / Beige |
| L | 218-376-3 | 32 - 34 cm | Beige / Beige |
| XL | 218-376-4 | 35 - 37 cm | Beige / Beige |

HUMEROVIT COMP 375

Hersteller
Manufacturer

Gebrauchsanweisung beachten
Consult instructions for use

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Please read this general information carefully. It contains important details regarding the use of this medical device. For questions, please contact your doctor or medical supply store.

In case any serious incident related with the device occurs, report it to the manufacturer. If the device shows signs of damage, stop using it immediately and dispose of it.

INSTRUCTIONS FOR USE (EN)

HUMEROVIT COMP 375 - Compression

PRODUCT DESCRIPTION

The device has been designed to compress and support the shoulder. It is intended to ease pain and swelling, and stabilize the shoulder during activity and recovery. It provides compression and support of the shoulder joint with a compressive knitted sleeve. The device stabilizes the shoulder with an under-the-arm sling strap of the contralateral arm. The silicon pad massages the shoulder and stimulates blood circulation.

Primary materials: polyamide, polyester

Pad: silicone

The device is designed for use on the left and right arm.

This device is intended for single patient multiple use.

INDICATIONS

- post-traumatic conditions / soft tissue injuries (e.g. dislocations, muscle strains, contusions)
- tendinopathies and overuse conditions (e.g. bursitis)
- post-traumatic or post-operative swelling (as directed by a clinician)
- support during physical activity or rehabilitation (e.g. myalgia, muscle insufficiency)

CONTRA-INDICATIONS

- Material hypersensitivity
- Open wounds at the affected shoulder

APPLICATION

1. Measure Your Upper Arm

- Measure the circumference of your upper arm.
- Select the correct size using the size chart on the packaging.

2. Put on the Brace

- Undo the Velcro strap.
- Insert your arm into the brace and position it over the affected shoulder.
- A removable Velcro pad can be added or removed to adjust the amount of pressure on your shoulder.

3. Secure the Strap

- Wrap the strap under the opposite armpit.
- Tighten the Velcro strap around your chest for a snug fit.
- Use the image on the packaging as a reference for correct positioning.

WARNINGS

- Contact your doctor or your medical supply store if complications arise.
- The first application and adjustment should be done by your doctor or your medical supply store.
- To maintain product safety and effectiveness, follow the device care instructions. Failure to comply may impair the functionality of the orthosis and result in potential health risks. Do not use cleaning methods other than those specified.

CARE

- Clean with a mild detergent at a maximum temperature of 30°C.
- Ideally clean the textile by hand, otherwise use a laundry net when using the washing machine.
- Secure all fasteners before washing.
- Dry with a towel and do not dry in direct heat (e.g., sunlight, radiator, etc.).
- Do not iron or dry clean.
- Do not bleach or use washing sodas.

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