Orpedo GmbH
Stieffring 2
Berlin D-13627

www.orpedo.com
+49 30 88 71 20 200
info@orpedo.com

Größe / Size	REF	Knöchel / Ankle
S	618-670-1	15 - 19 cm
M	618-670-2	20 - 23 cm
L	618-670-3	24 - 28 cm
XL	618-670-4	29 - 33 cm

ACHILLOVIT COMP 670

Hersteller
Manufacturer

Gebrauchsanweisung beachten
Consult instructions for use

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Please read this general information carefully. It contains important details regarding the use of this medical device. For questions, please contact your doctor or medical supply store.

In case any serious incident related with the device occurs, report it to the manufacturer. If the device shows signs of damage, stop using it immediately and dispose of it.

INSTRUCTIONS FOR USE (EN)

ACHILLOVIT COMP 670 – Compression

PRODUCT DESCRIPTION

The device has been designed to compress and support the heel and ankle. It is intended to ease pain and swelling, and stabilize the heel and ankle during activity and recovery.

The device is a soft knitted sleeve with a double-sided elastic flat knit technique to ensure an optimal fit and flexibility. An integrated silicone pad supports the tendon area in the foot.

Primary materials: lycra (elastane), polyester, polyamide
Heel pad wedge: silicone

The device is designed for use on the left and right foot.
This device is intended for single patient multiple use.

INDICATIONS

- post-traumatic conditions / soft tissue injuries (e.g. dislocations, muscle strains, contusions, ruptures)
- tendinopathies and overuse conditions (e.g. calcaneal spur)
- post-traumatic or post-operative swelling (as directed by a clinician)
- support during physical activity or rehabilitation (e.g. achilles tendon dysfunction, prophylaxis)

CONTRA-INDICATIONS

- Material hypersensitivity
- Open wounds at the affected heel and ankle

APPLICATION

1. Measure Your Ankle

- Measure the circumference of your ankle.
- Select the correct size using the size chart on the packaging.

2. Optional: Use the Assisting Sleeve

- To make it easier to put on the brace, place the assisting sleeve over your foot first.
- After positioning the brace correctly, pull the assisting sleeve out to remove it.

3. Put on the Brace

- Wear the brace like a sock.
- Ensure the logo is on the top side and your heel is positioned correctly.
- The soft pad should rest just above the heel.

4. Optional: Shoe heel Cushions

- If wearing the brace with shoes, add one heel cushion to each shoe to lift your heel.
- Do not place the heel cushion inside the brace!

WARNINGS

- Contact your doctor or your medical supply store if complications arise.
- The first application and adjustment should be done by your doctor or your medical supply store.
- To maintain product safety and effectiveness, follow the device care instructions. Failure to comply may impair the functionality of the orthosis and result in potential health risks. Do not use cleaning methods other than those specified.

CARE

- Clean with a mild detergent at a maximum temperature of 30°C.
- Ideally clean the textile by hand, otherwise use a laundry net when using the washing machine.
- Dry with a towel and do not dry in direct heat (e.g., sunlight, radiator, etc.).
- Do not iron or dry clean.
- Do not bleach or use washing sodas.

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