Orpedo GmbH  
Stieffring 2  
Berlin D-13627

www.orpedo.com  
+49 30 88 71 20 200  
info@orpedo.com

## EPIVIT FLEX 500

Hersteller  
Manufacturer

Gebrauchsanweisung beachten  
Consult instructions for use

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Please read this general information carefully. It contains important details regarding the use of this medical device. For questions, please contact your doctor or medical supply store.

In case any serious incident related with the device occurs, report it to the manufacturer. If the device shows signs of damage, stop using it immediately and dispose of it.

# INSTRUCTIONS FOR USE (EN)

## EPIVIT FLEX 500 - Compression

### PRODUCT DESCRIPTION

The device has been designed to compress and support the forearm. It is intended to ease pain and swelling, and stabilize the forearm during activity and recovery. The pressure pad reduces tension at the tendon insertion point. The device is secured with an adjustable Velcro strap to ensure precise compression and fit. The device is universally sized and can be worn on either arm.

Primary materials: polyamide, polyester, elastane  
Pad: silicone

The device is designed for use on the left and right arm.  
This device is intended for single patient multiple use.

### INDICATIONS

- post-traumatic conditions / soft tissue injuries (e.g. dislocations, muscle strains, contusions)
- tendinopathies and overuse conditions (e.g. epicondylitis (tennis elbow or golfer's elbow))
- post-traumatic or post-operative swelling ( as directed by a clinician)
- support during physical activity or rehabilitation

### CONTRA-INDICATIONS

- Material hypersensitivity
- Open wounds at the affected forearm

### APPLICATION

1. Prepare the Brace
  - Undo the Velcro strap.
2. Put on the Brace
  - Slide the brace over the affected part of your arm below the elbow.
  - If necessary, adjust the attached pad so that it is placed over the affected area.
3. Wrap the strap securely for a snug, comfortable fit.

### WARNINGS

- Contact your doctor or your medical supply store if complications arise.
- The first application and adjustment should be done by your doctor or your medical supply store.
- To maintain product safety and effectiveness, follow the device care instructions. Failure to comply may impair the functionality of the orthosis and result in potential health risks. Do not use cleaning methods other than those specified.

### CARE

- Clean with a mild detergent at a maximum temperature of 30°C.
- The silicone pad must be cleaned with a wet wipe.
- Ideally clean the textile by hand, otherwise use a laundry net when using the washing machine.
- Dry with a towel and do not dry in direct heat (e.g., sunlight, radiator, etc.).
- Do not iron or dry clean.
- Do not bleach or use washing sodas.

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