Orpedo GmbH  
Stieffring 2  
Berlin D-13627

www.orpedo.com  
+49 30 88 71 20 200  
info@orpedo.com

Größe / Size	REF	Knöchel / Ankle
S	618-615-1	15 - 20 cm
M	618-615-2	21 - 26 cm
L	618-615-3	27 - 33 cm

# PEDIVIT 615

Hersteller  
Manufacturer

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Please read this general information carefully. It contains important details regarding the use of this medical device. For questions, please contact your doctor or medical supply store.

In case any serious incident related with the device occurs, report it to the manufacturer. If the device shows signs of damage, stop using it immediately and dispose of it.

# INSTRUCTIONS FOR USE (EN)

**PEDIVIT 615** - Motion restriction

## PRODUCT DESCRIPTION

The device has been designed to restrict the motion of the foot. It is intended to protect the foot, ease pain and support the foot during recovery. It consists of two straps that wrap around the lower leg and midfoot. It supports weakened muscles (e.g., after stroke or peripheral nerve injury) and prevents stumbling. The material is soft and durable. The device promotes heel strike and helps to prevent foot drop.

Primary materials: elastane, polyester, polyamide

The device is designed for use on the left and right foot.

This device is intended for single patient multiple use.

## INDICATIONS

- post-operative conditions requiring motion restriction
- post-traumatic conditions requiring motion restriction (e.g. ligamentous injuries, nerve lesions (resulting in foot drop) or sprains)
- overuse or repetitive strain conditions
- degenerative or inflammatory conditions
- support during functional rehabilitation or physical activity (e.g. fatigue-induced paresis)

## CONTRA-INDICATIONS

- Material hypersensitivity
- Open wounds at the affected foot

## APPLICATION

1. Measure Your Ankle
  - Measure the circumference of your ankle.
  - Use the size chart on the packaging to select the correct size.
2. Fit the Ankle Brace
  - Open the Velcro strap and place the ankle brace (with logo) around your ankle.
  - Ensure the lace holes face forward and the Orpedo logo faces backwards.
  - Tighten the straps for a snug fit.
3. Attach the Foot Brace (Without Shoes):
  - Open the Velcro strap and place the foot brace (without logo) around your foot, with the Velcro straps on top.
  - Thread the lace through the lace holes of the ankle brace and hook it into the metal hooks.
4. Attach the Foot Brace (With Shoes):
  - Use the additional lace provided.
  - Pass it through the lowest eyelet of your shoe, then through the lace holes of the ankle brace, and hook it into the metal hooks.
5. Set the Foot Angle
  - Choose the desired foot angle.
  - Tie a secure knot.

## WARNINGS

- Contact your doctor or your medical supply store if complications arise.
- The first application and adjustment should be done by your doctor or your medical supply store.
- To maintain product safety and effectiveness, follow the device care instructions. Failure to comply may impair the functionality of the orthosis and result in potential health risks. Do not use cleaning methods other than those specified.

## CARE

- Clean with a mild detergent at a maximum temperature of 30°C.
- Secure all fasteners before washing.
- Ideally clean the textile by hand, otherwise use a laundry net when using the washing machine.
- Dry with a towel and do not dry in direct heat (e.g., sunlight, radiator, etc.).
- Do not iron or dry clean.
- Do not bleach or use washing sodas.

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