Orpedo GmbH  
Stieffring 2  
Berlin D-13627

www.orpedo.com  
+49 30 88 71 20 200  
info@orpedo.com

Größe / Size	REF	Taille / Waist
S	411-400-1	60 - 75 cm
M	411-400-2	76 - 90 cm
L	411-400-3	91 - 110 cm
XL	411-400-4	111 - 130 cm
XXL	411-400-5	131 - 150 cm

## LUMBOVIT DORSAL 400

Hersteller  
Manufacturer

Gebrauchsanweisung beachten  
Consult instructions for use

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Please read this general information carefully. It contains important details regarding the use of this medical device. For questions, please contact your doctor or medical supply store.

In case any serious incident related with the device occurs, report it to the manufacturer. If the device shows signs of damage, stop using it immediately and dispose of it.

# INSTRUCTIONS FOR USE (EN)

## LUMBOVIT DORSAL 400 - Posture correction

### PRODUCT DESCRIPTION

The device has been designed to correct the posture of the back. It is intended to ease pain, improve posture, and support healing of the back, trunk or spine. The device has a belt with elastic straps and removable stays to limit bending and stretching. The stays can be removed as recovery progresses in consultation with your doctor. Adjustable shoulder straps help to position the device.

Primary materials: neoprene, polyester. Stay: aluminum, steel  
This device is intended for single patient multiple use.

### INDICATIONS

- post-operative (spinal) support (as prescribed by clinician)
- post-traumatic conditions (e.g. fractures, muscle/ligament strains, dislocations)
- chronic or acute (back) pain (e.g. lumbosacral radiculopathy, prolapse, spinal stenosis)
- degenerative spinal disorders
- muscular weakness or imbalance requiring posture corrections (support in postural deformities) (e.g. lumbago)
- support in case of pain or injury during physical activity or occupational load (e.g. spondylolisthesis)

### CONTRA-INDICATIONS

- Material hypersensitivity
- Open wounds at the back

### APPLICATION

1. Measure Your Waist
  - Measure the circumference of your waist.
  - Select the correct size using the size chart on the packaging.
2. Optional: Insert Metal Stays
  - For additional support, insert the metal stays into the brace.
3. Position the Brace
  - Place the brace over your lower back.
  - Ensure it is centered on your spine.
  - Pull both sides of the belt and fasten them together over your abdomen.
4. Adjust the Tension Straps
  - Pull both small tension straps until the desired tension is achieved.
  - Attach them to the front belt.
  - Tighten evenly for a comfortable, supportive fit.
5. Secure the Shoulder Straps
  - Pull the upper straps over your shoulders and guide them under your armpits.
  - Cross the straps behind your back.
  - Pull both sides until the desired tension on your shoulders is achieved.
  - Secure the straps together over your abdomen.

### WARNINGS

- Contact your doctor or your medical supply store if complications arise.
- The first application and adjustment should be done by your doctor or your medical supply store.
- To maintain product safety and effectiveness, follow the device care instructions. Failure to comply may impair the functionality of the orthosis and result in potential health risks. Do not use cleaning methods other than those specified.

### CARE

- Clean with a mild detergent at a maximum temperature of 30°C.
- Remove the stays and secure all fasteners before washing.
- Dry with a towel and do not dry in direct heat (e.g., sunlight, radiator, etc.).
- Do not iron or dry clean.
- Do not bleach or use washing sodas.

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