Orpedo GmbH  
Stieffring 2  
Berlin D-13627

www.orpedo.com  
+49 30 88 71 20 200  
info@orpedo.com

Größe / Size	REF	Taille / Waist
S	418-512-1	60 - 75 cm
M	418-512-2	76 - 90 cm
L	418-512-3	91 - 110 cm
XL	418-512-4	111 - 130 cm

## LUMBOVIT 300b

Hersteller  
Manufacturer

Gebrauchsanweisung beachten  
Consult instructions for use

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Please read this general information carefully. It contains important details regarding the use of this medical device. For questions, please contact your doctor or medical supply store.

In case any serious incident related with the device occurs, report it to the manufacturer. If the device shows signs of damage, stop using it immediately and dispose of it.

# INSTRUCTIONS FOR USE (EN)

## LUMBOVIT 300b - Posture correction

### PRODUCT DESCRIPTION

The device has been designed to correct the posture of the lower back. It is intended to ease pain, improve posture, and support healing of the back, trunk or spine. The device has a compression belt and built-in aluminum splints. With adjustable pull-straps the fit can be customized. Its perforated neoprene retains therapeutic warmth.

Primary materials: neoprene, polyester

Splint: aluminum

Pad: silicone

This device is intended for single patient multiple use.

### INDICATIONS

- post-operative (spinal) support (as prescribed by clinician)
- post-traumatic conditions (e.g. fractures, muscle/ligament strains, dislocations)
- chronic or acute (back) pain (e.g. sciatica, spinal stenosis)
- degenerative spinal disorders (e.g. spondylosis)
- muscular weakness or imbalance requiring posture corrections (support in postural deformities)
- support in case of pain or injury during physical activity or occupational load (e.g. rheumatic pain)

### CONTRA-INDICATIONS

- Material hypersensitivity
- Open wounds at the lower back

### APPLICATION

#### 1. Measure Your Waist

- Measure the circumference of your waist.
- Select the correct size using the size chart on the packaging.

#### 2. Position the Brace

- Place the brace over your lower back.
- Ensure it is centered on your spine.
- Pull both sides of the belt around your body and fasten them together over your abdomen.

#### 3. Adjust the Tension Straps

- Pull both small tension straps until the desired tension is achieved.
- Attach them to the front belt.
- Tighten evenly for a comfortable, supportive fit.

### WARNINGS

- Contact your doctor or your medical supply store if complications arise.
- The first application and adjustment should be done by your doctor or your medical supply store.
- To maintain product safety and effectiveness, follow the device care instructions. Failure to comply may impair the functionality of the orthosis and result in potential health risks. Do not use cleaning methods other than those specified.

### CARE

- Clean with a mild detergent at a maximum temperature of 30°C.
- Secure all fasteners before washing.
- Dry with a towel and do not dry in direct heat (e.g., sunlight, radiator, etc.).
- Do not iron or dry clean.
- Do not bleach or use washing sodas.

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