Orpedo GmbH
Stieffring 2
Berlin D-13627

www.orpedo.com
+49 30 88 71 20 200
info@orpedo.com

GENUVIT COMFORT 310

Hersteller
Manufacturer

Gebrauchsanweisung beachten
Consult instructions for use

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Please read this general information carefully. It contains important details regarding the use of this medical device. For questions, please contact your doctor or medical supply store.

In case any serious incident related with the device occurs, report it to the manufacturer. If the device shows signs of damage, stop using it immediately and dispose of it.

INSTRUCTIONS FOR USE (EN)

GENUVIT COMFORT 310 - Motion restriction

PRODUCT DESCRIPTION

The device has been designed to restrict the motion of the knee. It is intended to protect the knee, ease pain and support the knee during recovery. The device limits bending and stretching of the knee with a controlled hinging mechanism. The simple mechanism does not require the use of tools. The struts can be adjusted in height to fit neatly. Velcro straps secure the device in position and prevent slippage.

Primary materials: cotton, polyamide, foam. Struts: aluminum

The device is designed for use on the left and right knee. This device is intended for single patient multiple use.

INDICATIONS

- post-operative conditions requiring motion restriction
- post-traumatic conditions requiring motion restriction (e.g. (collateral and cruciate) ligamentous injuries, nerve lesions, sprains, patellar fracture)
- overuse or repetitive strain conditions
- degenerative or inflammatory conditions (e.g. osteoarthritis)
- support during functional rehabilitation or physical activity (e.g. knee instability)

CONTRA-INDICATIONS

- Material hypersensitivity
- Open wounds at the affected knee

APPLICATION

1. Set the angle of the hinge
 - Push the small button on the disc.
 - Rotate the setting knob to the required setting.
 - Repeat on the other side.
2. Put on the brace
 - Loosen all Velcro straps.
 - Put the brace on, while sitting on the edge of a chair.
 - Close the strap above the bulge of the calf.
 - Make sure that the hinges are on the sides at the lower end of your knee cap.
3. Fasten the lower part
 - Push the red button on the strut of the lower part.
 - Slide until the Velcro strap is under the calf muscle.
 - Close the Velcro strap under the calf muscle.
4. Fasten the upper part
 - Close the lower Velcro strap on the thigh first.
 - Push the red button on the strut of the upper part.
 - Slide until the Velcro strap is halfway the thigh.
 - Close the Velcro strap on the thigh.
5. Optional: Add additional pads
 - To improve comfort insert the provided pads where needed.

WARNINGS

- Contact your doctor or your medical supply store if complications arise.
- The first application and adjustment should be done by your doctor or your medical supply store.
- To maintain product safety and effectiveness, follow the device care instructions. Failure to comply may impair the functionality of the orthosis and result in potential health risks. Do not use cleaning methods other than those specified.

CARE

- Clean with a mild detergent at a maximum temperature of 30°C.
- Remove the frame and secure all fasteners before washing.
- Dry with a towel and do not dry in direct heat (e.g., sunlight, radiator, etc.).
- Do not iron or dry clean.
- Do not bleach or use washing sodas.

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